



RECLINERS

COMMITTED TO QUALITY & SERVICE



The Porta Range

Our complete mobile and adjustable transfer chairs.



Our family of Porter chairs offer a Long-Term Postural and Pressure Management Seating Solution.

Suitable Environments



Home



Nursing &
Residential Homes



Healthcare



The Porter Chairs are designed for a range of users who require a combination of comfort, pressure management and posture control from their seating system.

The adjustability of the chairs and the range of complementary accessories mean that the chairs are suitable for users who require specific support and positioning to achieve good posture.

The chair can be used in single or multi-user environments.

The Porta Range should be selected for users:

- Who need to maintain a reasonable midline posture
- With neurological conditions which make them sensitive to discomfort or affect muscle tone so that support is required to combat fatigue
- For whom seat height for transfe is critical
- When posture pressure and comfort are important in equal measures
- Who require a wide range of positions for different activities through the day

The Porta chairs are suitable for users who have or are:

- Congenital conditions e.g.Cerebral Palsy
- Neurological conditions e.g. MS, MD, MND
- Lower limb amputation
- Early stage HD
- Recovering from a Stroke or have long terms effects of a stroke
- Learning difficulties
- Frail and/or elderly

Key Features...




Pressure Care Management

Using a Vapour Permeable Fabric with Four-way stretch allows alternative pressure care foams, cushions and gels to be incorporated in the seat offering low to high pressure care options to the user.



Tilt in Space System

Using a Tilt-in-Space mechanism with independent leg elevation and back recline option provides a change of orientation and thus redistributes pressure from one area to another and maintains physical angles at the hips, knees and ankles.

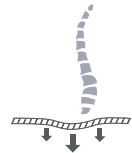

Postural Management

Using interchangeable back cushions and pressure seating options allows the user to achieve the most comfortable sitting positions and to achieve the correct posture.

Complete Tilt-In-Space Technology

Our Porta Range have an in-built Tilt-In-Space movement which is operated from the rear push handle allowing the chair to be gently reclined in a range of positions maintaining a constant back angle.



Independent Backrest Recline (manual only)

The backrest can be reclined independently providing a range of back angles. The back locks into position whenever the rear handle on the control bar is released. Enhanced posture and pressure management options are available by combining this movement with the legrest and footplate adjustment.

Independent Leglift Operation (manual only)

The manual Porta chairs come with an independent leglift facility which is operated by a clasp located on the side of the arm.



Manual Porta chair with Tilt-In-Space

Independent leg lift facility on Manual Mechanism only.

Robust and supportive arms for optimum user handling.

Complete 'Vapour Permeable' removable seat wrap and cushions.

Fully adjustable 'Tilt-In-Space' mechanism



Adjustable Footplate

The manual chairs come with an adjustable footplate facility.

You can alter the height by using the pull out pin positioned under the footboard. When raising user's feet you will need to adjust the position of the footplate and increase the depth to the required position.

The adjustable footboard can be removed completely if required.

Independent Extending Footrest

The independent extending footrest eliminates the need for manual adjustment to footplate through the footrest elevation.

This ultimately reduces the secondary users assistance whilst eliminating bending of the knees as the footplate is elevated into a rise position.



Operation and Handling - Manual Mechanism



Leg Lift Operation

The leg lift clasp, which is located on the outside of the arm, provides complete functionality, allowing the user to have their legs in the designed angle and position.



Recline Operation

The independent back recline is fully adjustable to suit the users specific requirements and can be operated by using the left-hand lever on the rear control bar.




Tilt-In-Space Operation

The manual TIS with control bar adjustment is easy to tilt by operating the right-hand lever on the rear of the chair. This facility caters for a variety of pressure requirements.

Reflexion Foam Seating

The below readings show a Tilt-In-Space seating position over a 15 minute test period of a user weight of 8 stone.

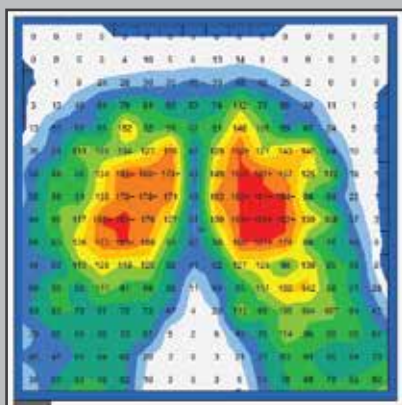

Conventional Foam Seat



Reflexion Foam Seat

Electric Porter chair with manual Tilt-In-Space

Powered by either a handset or paddle control, the electric porta offers a controlled Tilt-In-Space movement with electric leg lift and a manual Tilt-In-Space without leg lift.

To avoid electric cables we manufacture the chair with an in-built power pack (to maintain regular overnight charge required)

Porta Range Accessories



Lateral Back Cushion (Standard)

The cushion gives you moderate postural support to the lower back and trunk.



Arm Blockers

Our Vapour Permeable arm blockers provide simple lateral support where required.



Soft Headrest (Standard)

A deep, ultra soft pillow that conforms to the shape of head and shoulders.



Soft Profiled Headrest

A contoured soft pillow to fit shoulders, neck and providing a supportive head position.



Soft Butterfly Headrest

A soft pillow headrest which provides comfort along the curvature of the users neck.



Deep Contoured Headrest

Excellent support to users who suffer with severe head movements and neck injury.



Shallow Channel Legrest

Our shallow channel legrest offers lateral support to the users leg position.



Deep Channel Legrest

Our deep channel legrest supports a more central leg position for the user.



Adjustable Footplate (Standard)

A height adjustable footplate attaches to the legrest, once elevated to support feet.



Therapy Table

A wooden tray with low profiled edges to aid activities and leisure.



Chest Harness

The harness gives comfort and support without restricting active positioning.



Padded Pelvic Belt

The padded pelvic belt maintains position and is designed for user comfort.

Porta Range

www.recliners.org.uk

The Portaflex and Harmony Porta Size Availability.

Size Matrix	Measurements Shown in Inches.					
Seat Height	16	17	18	19	20	21 To Footplate
Back Height	26	28	30			
Seat Width	18	20	22			
Seat Depth	20	21	22	3" Adjustment		

*Any of the above sizes are available within the standard price



Portaflex



Harmony Porta



2 YEAR WARRANTY
ON ALL PORTER
CHAIRS AS STANDARD

DESIGN * CRAFT * QUALITY

Recliners, Old Griffin Field, Windsor Street, Pentre, Rhondda Cynon Taff, South Wales, CF41 7JJ
Telephone: +44 (0) 1443 431 000 Fax: +44 (0) 1443 440 923 Email: info@recliners.org.uk

www.recliners.org.uk